


1-1-1

rAyII

Migraine

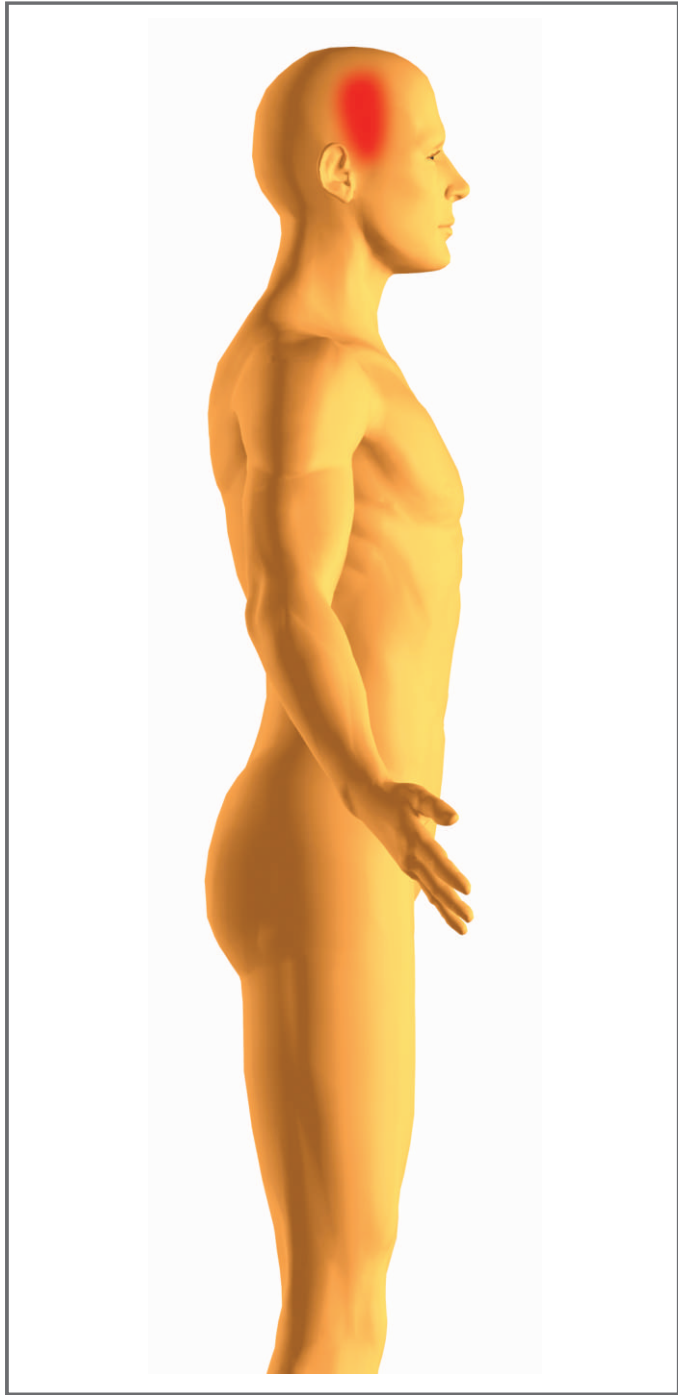

rAyII

Migraine (Spring)

rAyI/0:6






rTyII/0:c



rAyII

Migraine (Summer)

<i>rAyI/0:6</i>	<i>rAyIII/0:6</i>	<i>rTxIII/1:c</i>
		

rAyII

Migraine (Fall)

lTxI/1:(6)



lAxII/2:c



rAyII Migraine (Winter)

<i>lAxII/2:c</i>	<i>rAyII/0:c</i>
	

rAyII

Migraine (Dull Pain)


rTxIII/1:c



lTxI/1:(c)



rAyII Migraine (Muscle Pain)

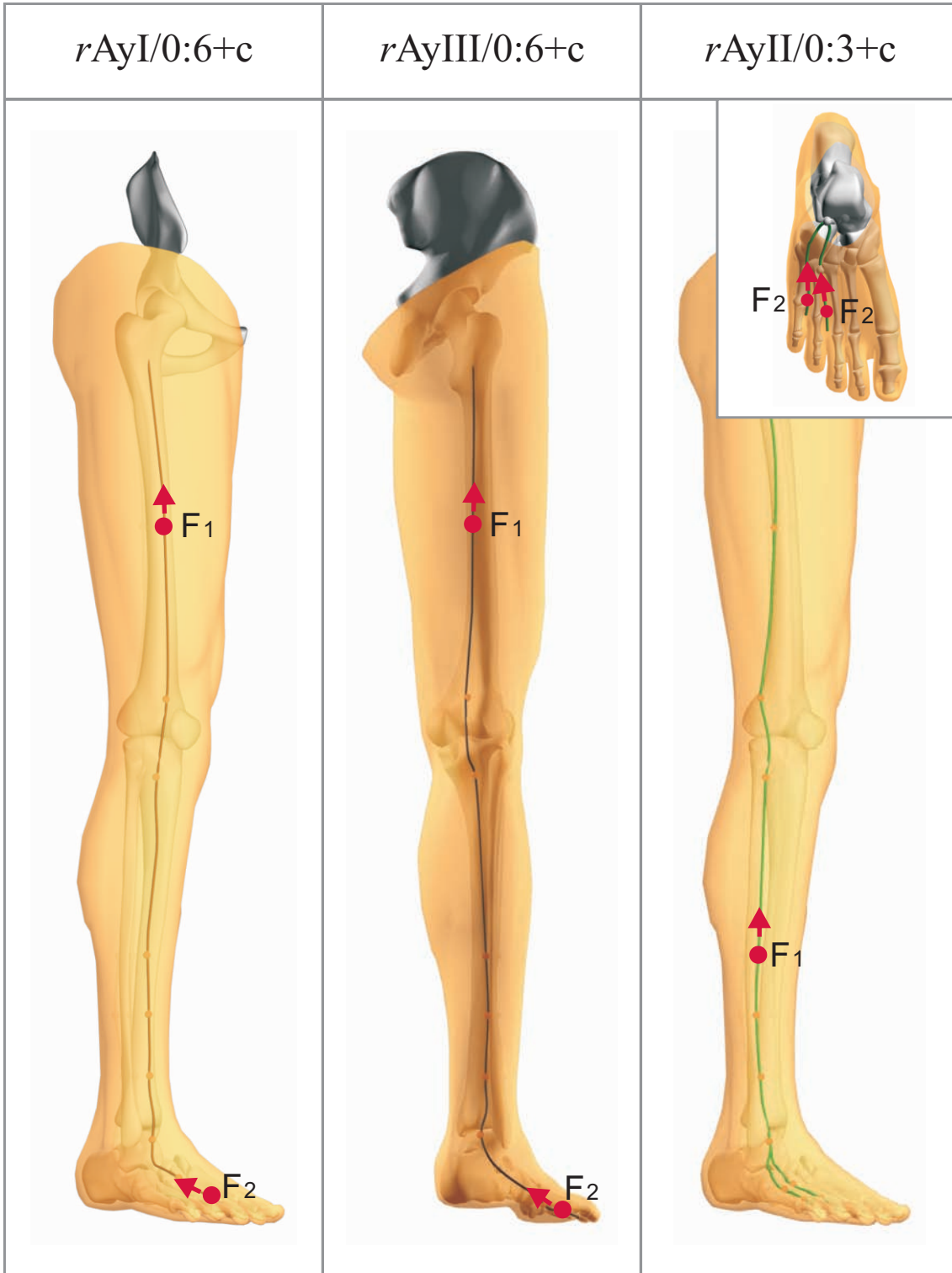

rAyII

Migraine

Spring

rAyI/0:6

C point not used, F point "6" using No. 3 head, stimulate proximally.

rTyII/0:c

C point not used, F point "c" using No. 2 head, stimulate distally.

Summer

rAyI/0:6

C point not used, F point "6" using No. 3 head, stimulate proximally.

rAyIII/0:6

C point not used, F point "6" using No. 3 head, stimulate proximally.

rTxIII/1:c

C point "1" using No. 1 head, F point "c" using No. 2 head, stimulate proximally.

Fall

ITxI/1:(6)

C point "1" using No. 3 head, F point "6" using No. 3 head, stimulate distally.

IAxII/2:c

C point "2" using No. 3 head, F point "c" using No. 2 head, stimulate distally.

Winter

IAxII/2:c

C point "2" using No. 3 head, F point "c" using No. 2 head, stimulate distally.

rAyII/0:c

C point not used, F point "c" using No. 2 head, stimulate proximally.

Dull Pain

rTxIII/1:c

C point "1" using No. 1 head, F point "c" using No. 2 head, stimulate proximally.

ITxI/1:(c)

C point "1" using No. 3 head, F point "c" using No. 2 head, stimulate distally.

Muscle Pain

rAyI/0:6+c

C point not used, F point "6" using No. 3 head, "c" using No. 2 head, stimulate proximally.

rAyIII/0:6+c

C point not used, F point "6" using No. 3 head, "c" using No. 2 head, stimulate proximally.

rAyII/0:3+c

C point not used, F point "3" using No. 3 head, "c" using No. 2 head, stimulate proximally.